SABOURAUD CAF (1g/L) AGAR

Selective medium for yeasts and moulds isolation.

TYPICAL FORMULA (g/l)

Peptomycol	10.0
Glucose	40.0
Chloramphenicol	1.0
Agar	15.0
Final pH 5.6 ± 0.2	

DESCRIPTION

SABOURAUD CAF (1g/L) AGAR is a selective medium for yeasts and moulds isolation from clinical samples.

PRINCIPLE

Peptomycol is a satisfying source of nitrogen for the growth of fungi and it contains a mixture of peptides and of free amino acids. The high glucose concentration and the acid pH make this medium selective for fungi: particularly the acid pH inhibits the bacterial growth, except the acidophili ones. The presence of chloramphenicol inhibits the bacterial contamination.

PREPARATION

Suspend 66 g of powder in 1 L of distilled or deionized water. Heat until completely dissolved. Sterilized in autoclave at 118°C for 15 minutes. Dispense in petri dishes.

TECHNIQUE

Streak the sample to examine onto the entire surface using a sterile loop. Incubate the plates at 30+/-2°C for 18-48 hours.

INTERPRETATION OF RESULTS

Observe for the formation of colonies.

STORAGE

10-30°C away from light, until the expiry date on the label or until signs of deterioration or contamination are evident.

WARNING AND PRECAUTIONS

The product is not classified as hazardous by current legislation and does not contain harmful substances in concentrations of ≥1%. The product is designed for *in vitro* diagnostic use and must be used only by properly trained operators.

DISPOSAL of WASTE

Disposal of waste must be carried out according to the national and local regulations in force.

REFERENCES

1. APHA (1963). Diagnostic procedures and reagents.
2. Booth, C. (1971) Methods in microbiology, Vol. 4, London: Accademy press.

Liofilchem s.r.l.

Via Scozia, Zona industriale - 64026 Roseto degli Abruzzi (Te) Italy - Tel. +39.0858930745 - Fax +39.0858930330
Web site: <http://www.liofilchem.net> - E-mail: liofilchem@liofilchem.net

PRODUCT SPECIFICATIONS

NAME

SABOURAUD CAF (1g/L) AGAR

PRESENTATION

Dehydrated medium

STORAGE

10-30°C

PACKAGING

Code	Content	Packaging
611203	500 g	500 g of powder in plastic bottle

pH OF THE MEDIUM

5.6 ± 0.2

USE

SABOURAUD CAF (1g/L) AGAR is a selective medium for yeasts and moulds isolation from clinical samples.

TECHNIQUE

Refer to technical sheet of the product.

APPEARANCE OF THE MEDIUM

Dehydrated medium

Appearance: free-flowing, homogeneous.

Colour: beige

Prepared medium

Appearance: slightly opalescent.

Colour: amber medium.

SHELF LIFE

4 years

QUALITY CONTROL

- Control of general characteristics, label and print
- Microbiological control
Inoculum for productivity: 10-100 UFC/ml
Inoculum for selectivity: 10⁴-10⁵ UFC/ml
Inoculum for specificity: ≤ 10⁴ UFC/ml
Incubation Conditions: 18-48 h at 30±2°C, in aerobiosis

Microorganisms		Growth
<i>Aspergillus niger</i>	ATCC 16404	Good
<i>Candida albicans</i>	ATCC 10231	Good
<i>Saccharomyces cerevisiae</i>	ATCC 9763	Good
<i>Trichophyton mentagrophytes</i>	ATCC 9533	Good
<i>Escherichia coli</i>	ATCC 25922	Inhibited

TABLE OF SYMBOLS

IVD In vitro diagnostic medical device	LOT Batch code	 Manufacturer	 Contains sufficient for <n> tests
REF Catalogue number	 Temperature limitation	 Use by	 Caution, consult accompanying documents

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LIOFILCHEM® S.r.l.