

Technical Data Sheet

Product information

Product code	11.0062
Product name	Campylobacter blood free selective agar
Short name	CCDA
Packing description (size, type, filling, amount)	90mm, Petri Dish, 18ml, 10pc(s)
Intended use	A selective medium for isolation of Campylobacter jejuni, C.coli and C.laridis.
Intended user	The in-vitro diagnostic device is intended for laboratory professional use only.

Storage Conditions

Storage temperature	2-12°C
Storage conditions	Keep the products in a low humidity environment, protected from moisture and light.
Shelf Life	See expiry date on the label (valid for closed packaging).
Shelf life after production	12 Weeks

Typical Composition *)

Ingredient	Amount of	Per liter
Nutrient Broth No. 2	25	g
Bacteriological charcoal	4	g
Casein hydrolysate	3	g
Sodium desoxycholate	1	g
Ferrous sulphate	0,25	g
Sodium pyruvate	0,25	g
Cefoperazone	0,032	g
Amphotericin B	0,01	g
Agar	12	g

*) Adjusted as required to meet performance standards.

Analytical Performance characteristics

pH	7,4+/- 0,2
Colour	black

Excellent
Environment
& Life

ONE¹
XDG

Clear No

Particles No

Microbiological Performance

Conditions:

Name	Incubation time	Incubation temp (°C)	Environment	Transfer	Incubation time	Incubation temp (°C)	Environment
inc-A	40 to 48 hours	35 +/-1	Camp. Gas				

Name	Microorganism	Identification number	Method	Specification	Electivity
inc-A	Escherichia coli	ATCC 25922 WDCM 00013 NCTC 12241	Selectivity of solid media	Complete inhibition	
inc-A	Staphylococcus aureus	ATCC 25923 WDCM 00034 NCTC 12981	Selectivity of solid media	Complete inhibition	
inc-A	Enterococcus faecalis	ATCC 29212 WDCM 00087 NCTC 12697	Selectivity of solid media	Complete inhibition	
inc-A	Campylobacter jejuni	ATCC 29428 WDCM 00156 NCTC 11322	Productivity of solid media	Good growth	slimey grey colonies

Microbiological Contamination

	Incubation time	Incubation temp (°C)	Transfer	Incubation time	Incubation temp (°C)	Specification
Inc-a	44 to 52 hours	30 +/- 1				No growth

Excellent
Environment
& Life

ONE¹
XDG



Contact:

Xebios Diagnostics Group GmbH
Tellerlingstrasse 49
45097 Düsseldorf, Germany
www.xebios.com
info@xebios.com
tel: +49 211 78 88 07 - 00

Date of creating: 01-11-2024

Version number: 1



Excellent
Environment
& Life

ONE¹
XDG